

"MINI CARRO": Support Seat for your car




INSTRUCTIONS

Before using a MINI CARRO

- To ensure safe use of MINI CARRO, please read and observe the instructions, particularly, the Safety Precautions.
- Please understand that the instructions must be changed according to the difference of the specifications.

KEEPING of the instructions

- A letter of guarantee, is included in the last page. Please keep carefully so that you can take out whenever you need.
- The instructions must be always with the product, when you hand over or lend it .

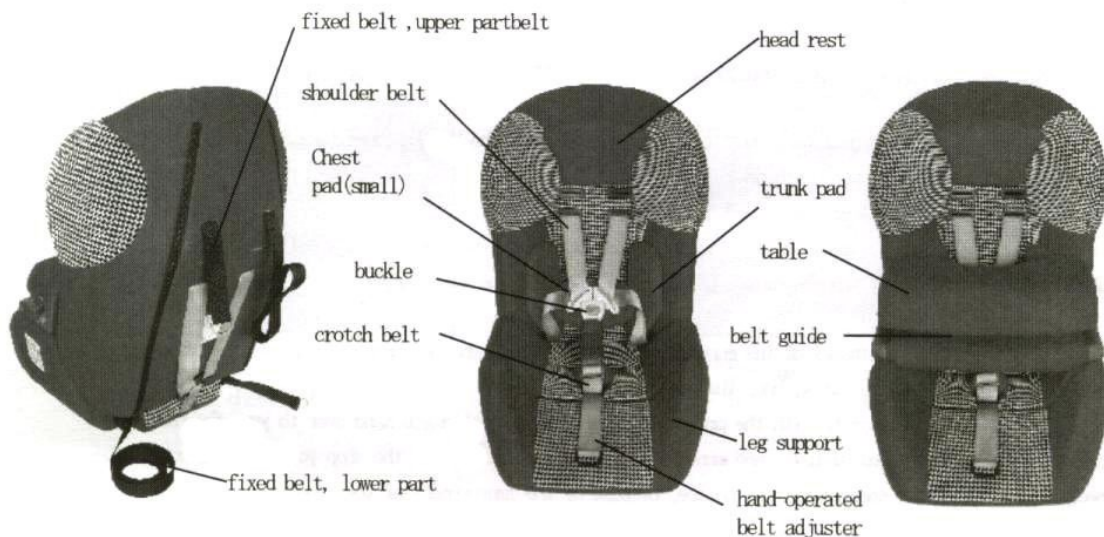

MINI CARRO is a support seat developed for disabled children to enjoy safety and comfortable drive on your car.

BEFORE USING MINI CARRO

Who can use?

- Age : 1 around 3 (depending on a user's physique. See the size table.)
- Weight: 9 below 18kg
- height: below 110cm

Do not use the following car seats for a MINI CARRO.

- a car seat without a seat belt
- a car seat fixed sideways, or backward
- a car seat of under 40cm in depth
- a car seat with an extreme dent on the seat, which leads to unstable fixing
- a car seat with a waist belt with ELR
- a car seat with a passive seat and 2 point system ELR

SAFETY PRECAUTIONS

Warning [This symbol denotes that improper handling poses a high risk of causing death or serious injury.]

- Do not use a car seat without a seat belt.
- Before attaching, make it sure that the seat belt is not twisted, and the buckles are locked.
- Do not use a car seat with an extreme dent on the seat, which leads to unstable fixing.
- There are various types of seat belts. Before attaching, make sure the type of your seat belt in your car instructions.
- MINI CARRO must be attached facing the front with the seat belt of your car.
- For your safety, do not keep using MINI CARRO after shocked by a strong collision, even if it looks with no damage.
- Do not leave a MINI CARRO in your car, but keep it in a trunk when you do not use. Hold it with a fixing belt in a car.
(In case of an accident, it is very dangerous)
- To protect a child from a burn with heated metal parts in your car, make sure the temperature of these parts in hot season.
- Do not panic in case of an emergency such as an accident. Unlock the buckle, then take a child out of the seat.
- Do not use without a cover and urethane.
- Do not take the parts apart or remodel of the product.
- Fix luggage or belongings firmly so that you are not injured with them in case of a collision.
- Take care if hard parts of the system are caught in the seat or the door, when you attach the junior seat to your car seat.
- Use soft materials(as urethane) of the manufacturer recommended when you change it, unless the efficiency falls.
- Do not leave the child in a car without the protector.
- The instructions must always be with the product. Do not forget it when you hand over to your friends.
- When you have any questions of using and mending ways, please contact with the shop you bought or with us.
- Specifications might be changed without notice, because of the improvement of the product.

THE WAY TO ADJUST

ORDERS to adjust

1. BEFORE the final fixing

First, loosen the adjustment nuts and the shoulder belt of a MINI CARRO on the back of the back seat.

Put the MINI CARRO temporarily on your car seat, and let the child sit on.

Set the chest pad and belts. Adjust the depth of the seat and the height of the back.

After pulling out the belt of the hand-operated belt adjuster fully, pull it back a little.

Decide the length of the shoulder belt and fix it to the tube on the back of the back seat.

After setting the length and the size of the belt, let the child get off.

Then adjust and fix to an adequate and final point.

2. Set the position of the slits for the shoulder belts

First have your child on the seat and pull out the belt from the slit of his or her shoulder height.

In case of changing the slit, take out the belt from the belt adjuster on the back of the backrest.

Follow the order 4 to fix the belt to the adjuster.

3. Adjustment of the length of the crotch belt (adjustment of the height of the chest pad)

The height of the chest pad is changed with adjusting the length of the crotch belt. Adjust the height carefully for the chest pad not to obstruct the child's neck.



adjusting the belt adjuster
under the chest pad



putting the rest part of the
belt into the chest pad

4. Adjustment of the depth of the seat


The depth of the seat depends on the length of the use's thigh on the seat, physical function and time to use. The standard type is set to sit with his legs stretching out. With the support of urethane step kit, he can sit with his knees bending.

The screws under the seat to adjust the front seat must be fixed up after adjustment.


You can fill the space between the front seat and the main seat.



screw to adjust the length of the seat

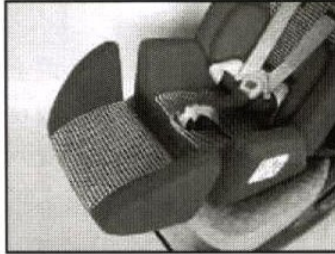

the seat expanded

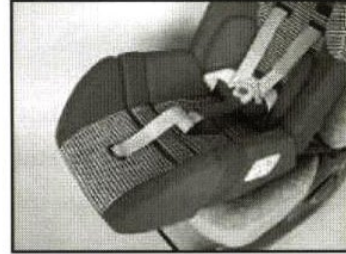

filling the space with an extra pad

5. Adjustment of the height of the head rest

Adjust freely using velcro on the back of the head rest.

Take care not to press the ears.

HOW TO ATTACH A MINI CARRO TO YOUR SEAT


Attaching a MINI CARRO to your car seat

1. Attach the junior seat firmly to your car seat with the fixed belts both on the upper and the lower parts of the MINI CARRO back seat.

In case of a separated car seat, use the upper and the lower fixed belts.

When you have a bench seat, fix it with the upper and the lower fixed belts turning vertically, or only with the upper belt turning around the head rest.

Please remember, MINI CARRO needs a seat belt of three-point-style to fix to your car seat of any kind.

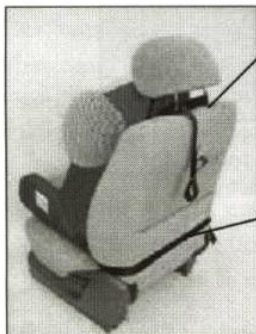

the fixed belt on the upper part of the back seat

Lift up the head rest of your car seat, adjust the length of the belt, and fix firmly

the fixed belt on the lower part of the back seat

Adjust the length of the belt at the lower part of the back seat of your car seat.



passing the belt through a D ring



turning and passing the belt between the two D rings, and pulling out to fix

USING YOUR MINI CARRO

Let's try MINI CARRO after adjusting and fixing to your car seat.

1. With one touch of a button, you can adjust the belt to get on and off.

MINI CARRO's non-escape system keeps your child safe, and it makes possible to adjust the length of the belt with one touch of a button.

Getting out: push the release button of the hand-operated belt adjuster on the inner part of the front seat, then the shoulder belt is released and the belt is pulled out.

Getting on: Shorten the shoulder belt by pulling the belt on the front seat.



pushing the release button
to pull out the belt



pulling out the belt of the
hand-operated belt adjuster to
shorten the shoulder belt

2. For the maximum safety of the user, when you install a MINI CARRO,
never forget passing the 3 point seat belt of your car through the table.

- After deciding the child's position with the 5 point seat belt of a MINI CARRO, pull out the 3 point seat belt of your car and pass it through the belt guide of the table. Refer the last part of the explanatory seal. Make sure the seat belt is not twisted and the belt is passed correctly through the belt guide before fixing the seat belt.

- Making sure with your hands that the table is fixed firmly. When it is loose, pull the shoulder seat belt of your car.

- The person in the same car must always pay attention to the table. When it becomes loose, fix it firmly again.



(1) Pass the 3 point seat belt of your car
through the belt guide of the table and fix
it firmly.



(2) Feeling the table with your hands and
adjust the seat belt to fix it firmly.

3. Do not recline the back seat too much.

Too much reclining may lead the seat belt useless and dangerous.

4. MINI CARRO itself does not ensure your safety in case of collision.

MINI CARRO is the support seat developed to support the body of the disabled who have difficulties to position on a car.


It is dangerous for the child to keep a lying position on your car, as the seat belt is of no use.

Do not forget to use the seat belt of your car after adjusting your MINI CARRO's belts and


OPTIONAL PARTS



extra pads 50m/m

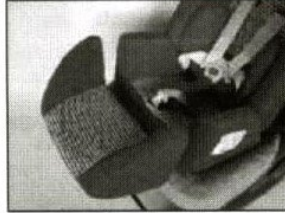


long seat with the seat and pads putting together

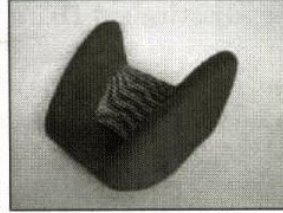

urethane step kit



urethane step kit setting in

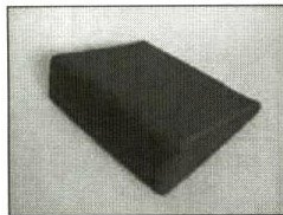

triangle mat





mat setting in



support pad

SIZES OF PARTS

whole width	420mm	width of the trunk	about 210mm
seat depth	240mm(without the front seat)	height of the back seat	650mm
	415mm(standard)	weight	9 kg
	465mm(with 50mm extra pad)	applied weight	9~under 18kg
	475mm(with urethane step kit)	applied age	1~around3

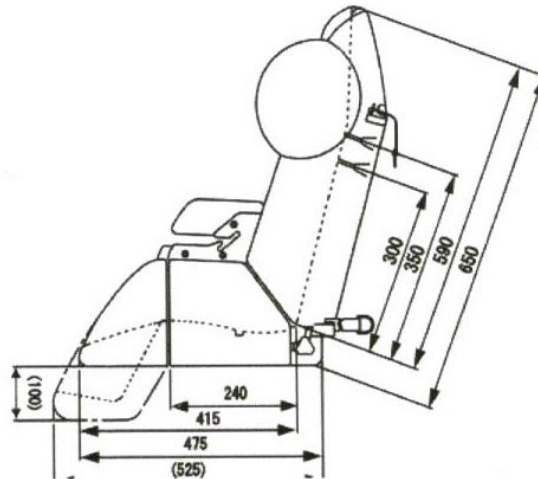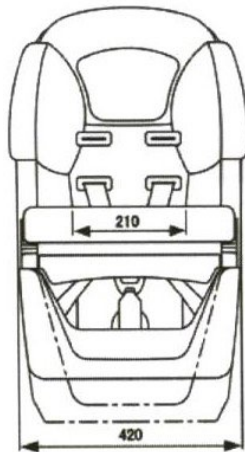


COVER REMOVE INSTRUCTIONS

Remove the covers as follows. (You can put them on in reverse order)




1 Take away the shoulder belt at two points on the back of the seat back.

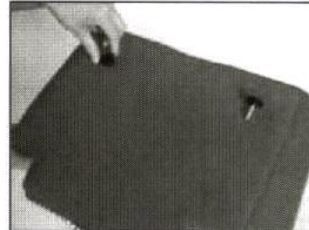

7 Take away 2 handle nuts on the back of the back cushion.



2 Push the button of the buckle on the chest pad and pull out the both tongue plates.

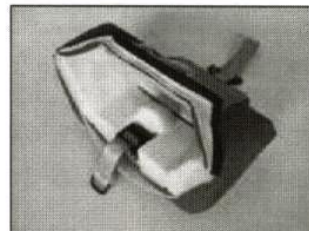

8 Take away the cover by peeling out the velcro on the front.




3 Take away the chest pad by pulling out the belt from the belt adjuster on the lower edge of the chest pad.




9 Take away the cover by peeling out the velcro on the back of the front cushion.



4 Take away the handle nuts L/R of the seat back and the cushion.

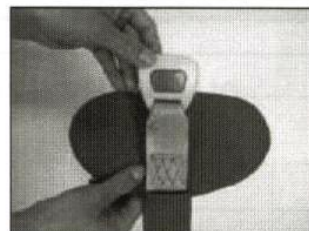


10 Pull out the buckle of the chest pad.

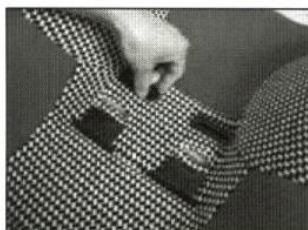

5 Pull out 4 belt crimps heavily.



11 In case of the urethane step kit, take out the cover peeling the velcro.

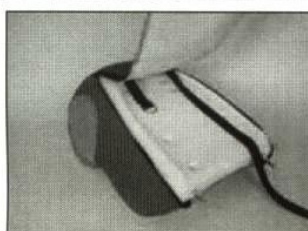

6 The cover is taken away by opening the zip fastener after peeling out the velcro.



12 You can wash the table Dry it wiping with a towel.

NOTE: 1 Wash with hands, do not use the washing machine.

2 If unable to take away, please make contact with a dealer.

LIMITED WARRANTY

1. The product is warranted free of charge during the period of a guarantee in case of the troubles caused by defects in material.
2. The product is warranted for a period of one year after purchase.
3. This warranty does not cover the following cases. You will bear the cost.
 - (a) failure and improper use
 - (b) modification
 - (c) damages by natural calamity
 - (d) the guarantee certificate without the name and the seal of the dealer where you purchased the product
 - (e) in case you do not show the guarantee certificate
4. This warranty is valid only in case the product is installed in Japan.
5. This guarantee certificate shall not limit your legal rights.

Maintenance Service

In the event of a problem during the period of the warranty, please contact the dealer where you purchased the product with the guarantee certificate.

CAUTION

GUARANTEE CERTIFICATE

GUARANTEE CERTIFICATE

model	MINI CARRO		
customer	address		
	name	TEL	
date of purchase			
dealer			
general agency			