



Moonlite sleep system



24-hour
postural care


JCM Customer Care Commitment



We are committed to taking care of our customers by providing a thorough after sales service and one-to-one customer assistance.

Our highly trained team of national representatives are on hand to provide free demonstrations, assessments and equipment set-ups to ensure your product is perfectly positioned for the user.

We pride ourselves in being able to offer a fantastic 3 year manufacturer's warranty as well as providing the option of service and maintenance visits by our trained Rehab Technology Suppliers to keep your product in perfect working order.

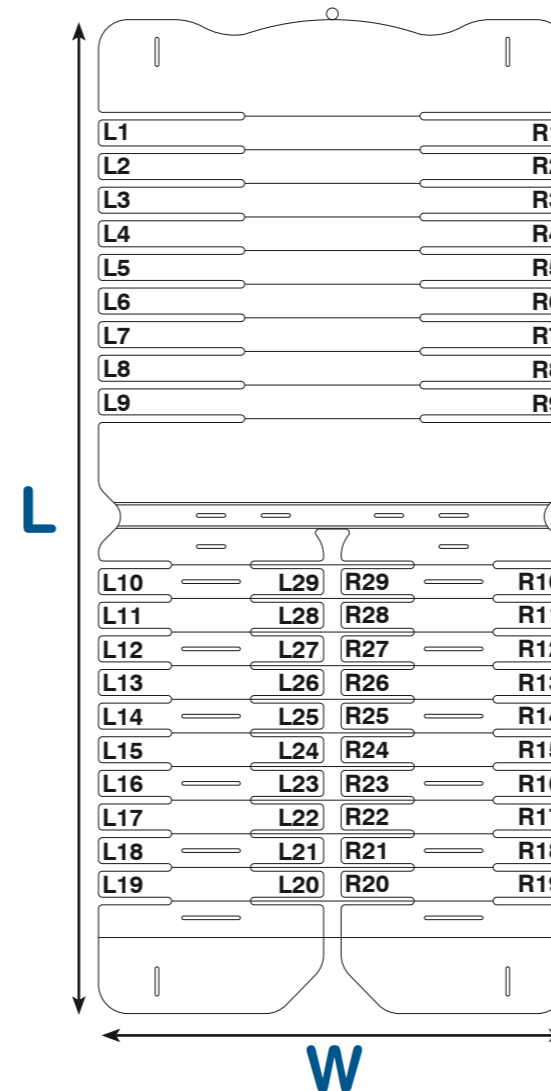

DIMENSIONS (inches)		
STANDARD BED		
L	Length	67
W	Width	31½
COT		
L	Length	47½
W	Width	22
CENTRE BASE		
L	Length	15½
W	Width	31½

An assessment chart is provided to ensure correct choice of positioning. The bed and the pads themselves are labelled to ensure you can repeat the desired support each and every time the system is set up. This is especially beneficial when tailoring a program for a child or when the system is used for a group of children.

We provide a trial service for our sleep system. Please call our office if you would like more information.

Please Note

All measurements given for indication only. Our representatives will assess your child to ensure the sleep system is fit for purpose both now and for the future.

www.jcmamerica.com™



JCM America™
521- D Uwharrie Court
Raleigh, NC 27606

Phone 919-831-9000
Fax 919-831-9100
Toll Free 888-831-9831
Email enquiries@jcmamerica.com
Website www.jcmamerica.com



A fresh approach to sleep system design, the Moonlite is flexible and easy to use whilst still providing uncompromised postural support

24-hour postural care

The Moonlite offers the ability to follow through with postural work completed throughout waking hours, enabling 24-hour postural care to be possible.

The system has a light, easy to clean plastic sub frame that can be flexed to profile the shape you require, supporting your client from head to toe. This means that the system can

function on its own or with any profile bed. The supports slide and lock onto the frame providing maximum side support and maintaining midline position. These supports are adjustable in angle and can be laid flat to make moving and handling the client in and out of bed easier without having to remove every bracket.

Can be fitted to a profile bed if required



Vast range of support options



support cushion option available

*actual cushion will be cream coloured



Knee Abduction

This system is used to maintain alignment of the legs in a supine lying position; providing midline symmetry and preventing windsweeping by securely supporting the knees. The knee cup size is carefully adjusted in assessment to precisely fit the individual, providing maximum comfort and support. Independent adjustment of the knee cups ensures the correct degree of abduction or adduction as required. The abduction system can either be used in conjunction with the Moonlite Sleep System or as an accessory on its own.

Function

The adjustability of the system, using lateral pads with a range of different sizes, allows for progressive alterations to be made to the individuals sleeping position, taking into account the changing needs throughout their life. The clear method of illustrating the system set-up also offers a documentation tool representing the pattern of change in the individuals sleeping position over time.



The Moonlite can be used for supine, prone and side lying



Handling

The brackets ensure secure postural support when fixed in position but can be easily re-positioned or folded flat to allow personal care to take place during the night, without having to dismantle the system.

Fold-Flat Support



The lightweight system can be folded down for transportation with the brackets still in place, alleviating the problem of having to reset the system every time it is moved. The system can also be supplied with a travel bag.



Cot & Adult sizes

3 Layers

①

Supports



The Moonlite frame and supports form the base of the system.

②

Pressure relieving foam & temperature regulating gel pads



The padded cover is made from pressure relieving foam and optional gel pads help regulate the users body temperature.


③

Stretch top sheet



A two way stretch top sheet completes the system.

Side Lying Positioning

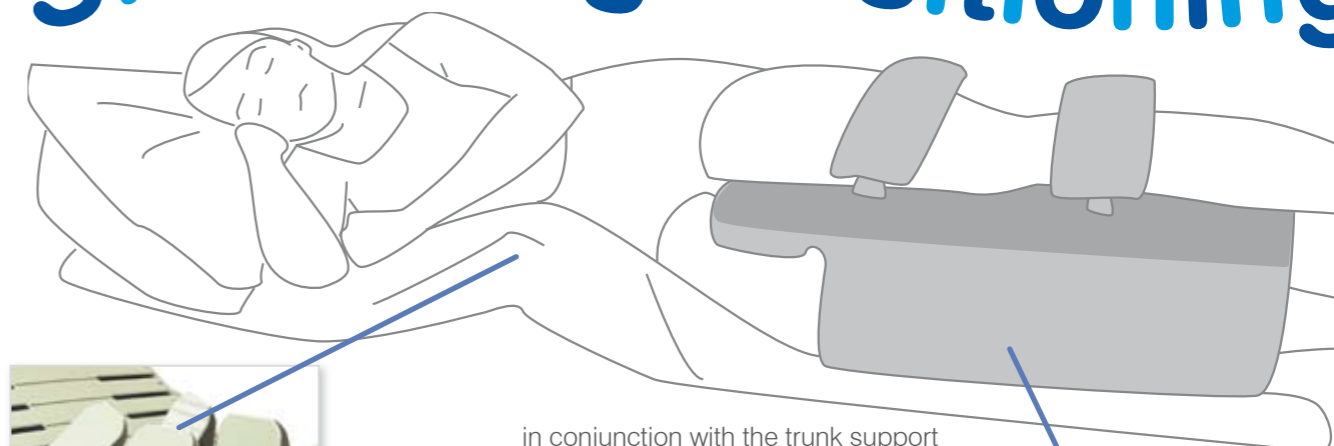

The leg support is designed to position and abduct the legs in side lying, free standing on top of the system it is easy to remove and replace when users need to be turned during the night. The top leg is fully supported which in turn maintains the position of the pelvis, this

in conjunction with the trunk support brackets hold the body in a mid line position and discourages rotation in the hips and trunk.

Available in different sizes and adjustable in both height and angle the side lying leg support solves a complex problem with minimum fuss. For more active users, the top of the support is slotted so that soft straps can be added to the system to help guide the position of the leg.



Side Lying Table